Ningbo Medsun Medical Co., Ltd.		
Title: IFU of Blood Lancet for Single Use		NO.: IFU-001
Page: 1/4	Edition/Revision: 01/01	Issue Date: 2020.06.10

[Product Name]: Blood Lancet for Single Use

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[The Combination of the Device with Product]: 设备与产品的结合

Model BYY、BYY1、BYY2、BYY3、BYY4、BYB used with lancing device.

Model XY1、XY2、XY3、XY4、XY1 Blade、XY2 Blade、XY3 Blade、XY4 Blade、XH1、XH2、XH3、XH4、XH5、XH6、XH1 Blade、XH2 Blade、XH3 Blade、XH4 Blade、XH5 Blade、XH6 Blade used independently.

BYY、BYY1、BYY2、BYY3、BYY4、BYB 与采血笔配合使用

XY1、XY2、XY3、XY4、XY1 Blade、XY2 Blade、XY3 Blade、XY4 Blade、XH1、XH2、XH3、XH4、XH5、XH6、XH1 Blade、XH2 Blade、XH3 Blade、XH4 Blade、XH5 Blade、XH6 Blade 型采血针单独使用

[Product Brief Introduction]: 产品简介

Model BYY、BYY1、BYY2、BYY3、BYY4、BYB blood lancet is composed with protective cap, solid needle and handle, a silicone coating is applied to the needle tip;

Model XY1、XY2、XY3、XY4、XY1 Blade、XY2 Blade、XY3 Blade、XY4 Blade blood lancet is composed with Protective cap、Slider、Needle、Short spring、Housing、Needle holder、Long spring、Back cover, a silicone coating is applied to the needle tip, except model XY3、XY4、XY3 Blade、XY4 Blade.

Model XH1、XH2、XH3、XH4、XH5、XH6、XH1 Blade、XH2 Blade、XH3 Blade、XH4 Blade、XH5 Blade、XH6 Blade blood lancet is composed with Springs、Back cover、Housing and Needle holder; Silicone coating is applied to the needle tip, except models XH3、XH4、XH6、XH3 Blade、XH4 Blade、XH6 Blade.

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Model BYY、BYY1、BYY2、BYY3、BYY4、BYB

1.Preparation before use

- a) Select proper specification of the lancet per request;
- b) Scrub and disinfect the test site with iodine or alcohol.

Ningbo Medsun Medical Co., Ltd.		
Title: IFU of Blood Lancet for Single Use		NO.: IFU-001
Page: 2/4	Edition/Revision: 01/01	Issue Date: 2020.06.10

2、Instructions

a) When collecting blood, install the lancet in the lancet holder, remove the protective cap of the lancet, and then cover the cap of lancing device. Then align the lancet with the sterilized area and press the button to complete. (See the lancing device manual for the operation method)

c) Once used, simply discard lancet into a sharps container or dispose per the local regulations.

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Model XY1、XY2、XY3、XY4、XY1 Blade、XY2 Blade、XY3 Blade、XY4 Blade

1.Preparation before use

- a) Select proper specification of the lancet per request;
- b) Scrub and disinfect the test site with iodine or alcohol.

2. Instructions

- 1) Remove the protective cap from the lancet;
- 2) Place end of lancet on the test site;
- 3) Push the lancet down against test site to activate the lancet mechanism;
- 4) Once used, simply discard lancet into a sharps container.

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1.Preparation before use

- a) Select proper specification of the lancet per request;
- b) Scrub and disinfect the test site with iodine or alcohol.

2. Instructions

- 1) Twist and remove the protective cap from the lancet;
- 2) Place end of lancet on the test site;
- 3) Push the lancet button down against test site to activate the lancet mechanism;
- 4) Once used, simply discard lancet into a sharps container.

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[Contraindications]: None **禁忌症: 无**

[Storage and transportation]: **存储和运输**

1. The lancets should be protected against collision and extrusion to avoid damage to the sealed package and the deformation of the needle body.
2. The products shall be stored in a dry, clean, and well ventilated environment.

Ningbo Medsun Medical Co., Ltd.		
Title: IFU of Blood Lancet for Single Use		NO.: IFU-001
Page: 4/4	Edition/Revision: 01/01	Issue Date: 2020.06.10

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Shanghai International Holding Corp. GmbH